



A 100% Australian owned company
supplying scientific laboratories throughout
Australia and overseas since 1987.



Put safety first with the Clicksmart® Blade-Flask blade remover!

Remove, contain, and dispose of scalpel blades in one click, one hand. The Blade Flask scalpel blade remover enables healthcare professionals to quickly and safely remove up to 100 scalpel blades.

- OSHA compliant with single-handed activation when used with the reusable Mounting Bracket (Sold Separately)
- Audible "CLICK™" sound signifies safe scalpel blade removal and containment
- Built-in counter with automatic shutoff when maximum capacity is reached
- Mount on a wall, bench, or trolley with the Clicksmart universal Mounting Bracket (product code QBRGEN)
- Puncture-proof sharps container which meets Australian Standard 4031:1992
- Compliant with Australian Standard AS 3825:2020 and sharps safety practices
- Compliant to International Standards and Regulations
- Comes under Engineering Controls as a passive-safety engineered device to prevent occupational injury and infection
- Compatible with most brands of general surgery blades and flat BP handles

Rowe Code HB0500

Blade-flask blade remover unit.

Buy 1 \$36.40 ea or Bulk buy 24+ \$32.76 ea

Rowe Code HM0501

Mounting Bracket

Buy 1 \$6.42 ea or Bulk buy 24+ \$5.78 ea



Watch the video



See over, for ease of use...

South Australia & NT

Ph: (08) 8186 0523

rowesa@rowe.com.au

Queensland

Ph: (07) 3376 9411

roweqld@rowe.com.au

Victoria & Tasmania

Ph: (03) 9701 7077

rowevic@rowe.com.au

New South Wales

Ph: (02) 9603 1205

rowensw@rowe.com.au

Western Australia

Ph: (08) 9302 1911

rowewa@rowe.com.au

PARTS OF YOUR BladeFLASK:


INDICATOR:
SHOWS WHICH WAY SHARP
EDGE MUST FACE.

THIS PART OF
THE SCALPEL
HANDLE IS
CALLED THE
TANG.

COUNTER

LANDING
PAD


MOUNTING
BRACKET

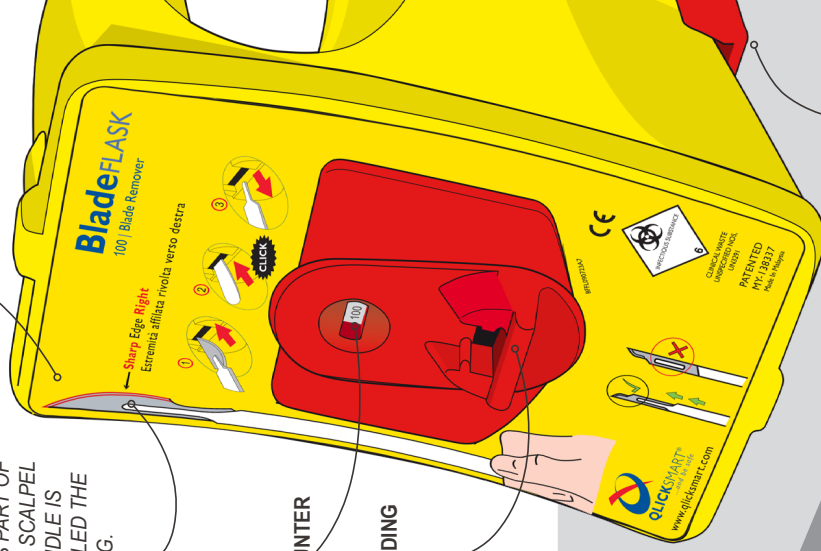

MOUNT TO WALL OR TROLLEY:

1. If you haven't mounted your BladeFLASK before, prepare mounting bracket*.

A BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM

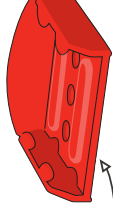


B OR USE SCREWS.

* See fitting template and instructions inside the bracket packaging.




2. BladeFLASK can be mounted on a wall or on a trolley or bench.

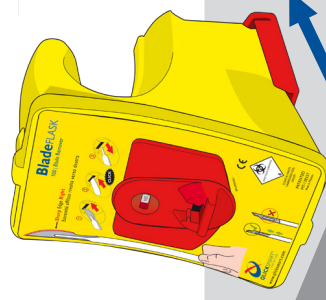


NOTE: BladeFLASK is mounted on Mounting Bracket during use.

REMOVE BLADE SINGLE-HANDED ACTION:


1. Place blade on landing pad with the sharp edge facing to the right (or 'tang' up).

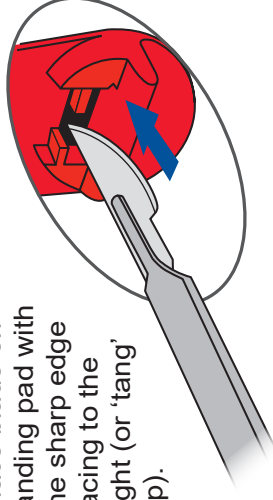

2. Push scalpel handle into BladeFLASK until you hear a click.

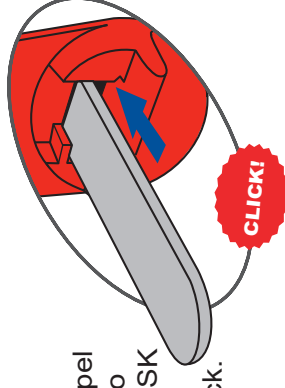

3. Remove handle.

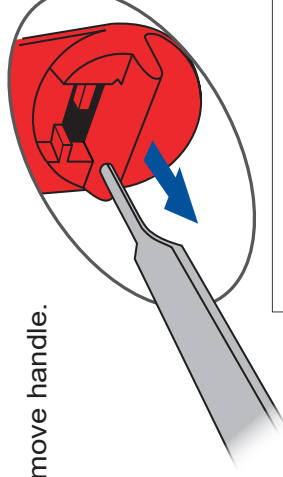

NOTE: BladeFLASK is disposed of as medical waste after 100 blades have been inserted.